


IRISS mediGrid for iPhone – Quick Start Guide


Start here



Install the app

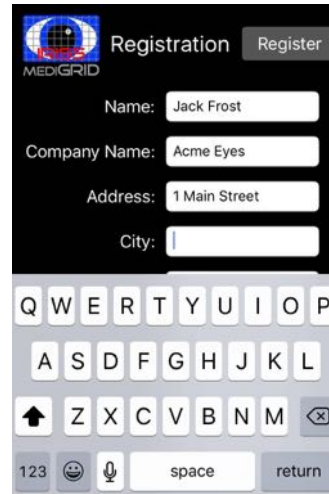

Register details for sticker delivery

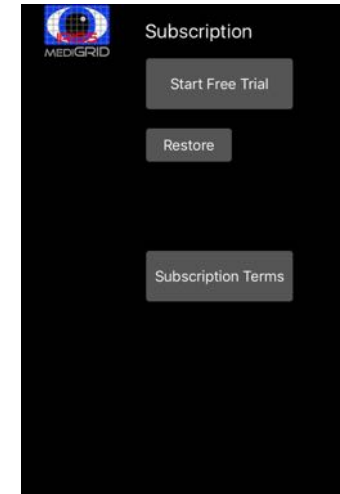


Touch **Start Free Trial** to begin the 1 month trial period

→



Receive stickers in the mail

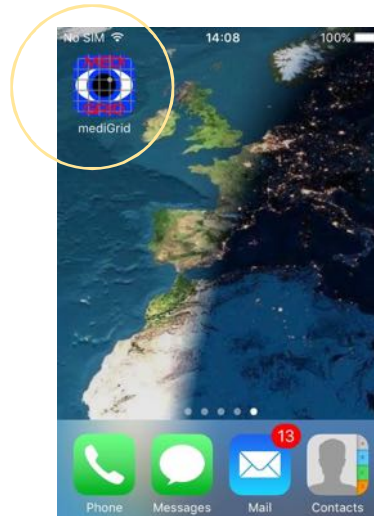


Start the app

→




Enter patient ID / reference


IRISS mediGrid for iPhone – Quick Start Guide

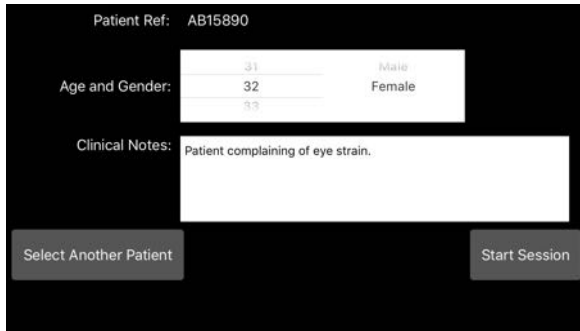


Enter age, gender & clinical notes then touch **Start Session**

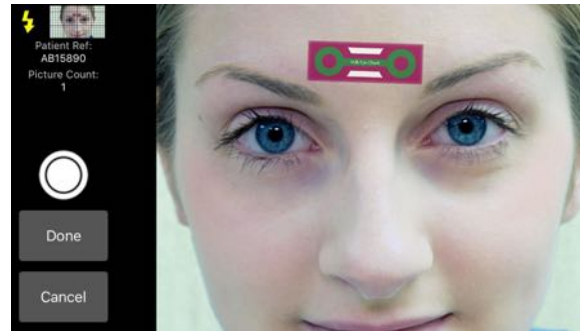

Apply sticker horizontally near the area to be photographed and take pictures

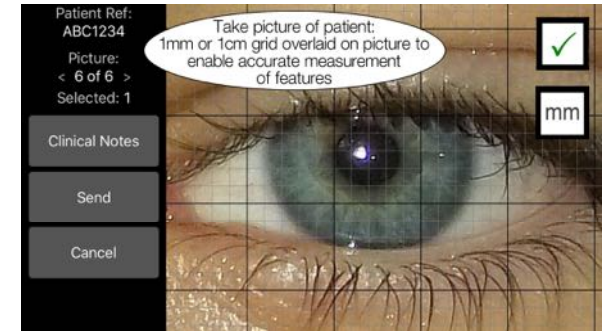

1mm or 1cm grid will appear and can be switched to other size

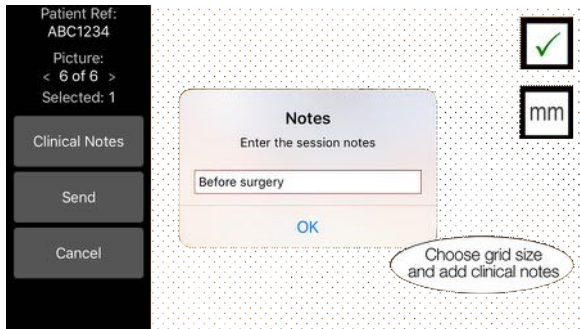

Select up to 3 pictures for each upload then touch **Send**

Pictures will arrive in your email inbox together with patient details and clinical notes